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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.


Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The knee walker is intended to be used as a mobility device for foot or ankle injuries where the affected area cannot bear weight or has limited precautions for bearing weight. The walker can be used by the patient themselves without the assistance of a third party.

PRODUCT DESCRIPTION

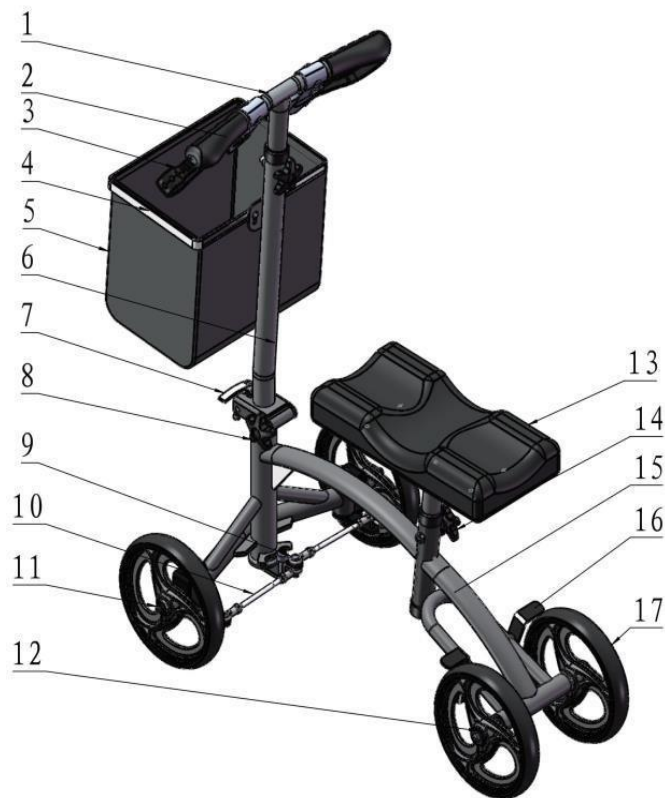- 1. T-shape Handle Assembly
- 2. Plastic Grip
- 3. Hand Brake
- 4. Welded Basket Holder
- 5. Cloth Basket with Cover
- 6. Foldable Upper Bar Assembly
- 7. Spring Quick Release
- 8. M10 Rotary Knob (Star-shaped)
- 9. Hexagon Socket Head Cap Screw M6*35 (standard)
- 10. Joint Bearing Assembly
- 11. M8 Self-locking Nut
- 12. Dust cover
- 13. Knee Platform Assembly
- 14. M8 Knob Screw
- 15. Main Frame
- 16. Rear Brake Mechanism
- 17. Eight-inch Wheel Assembly

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 136 kg.
- Check all components are secured before use
- Make sure that the product is in good working order.
- Visually check the product
- Before each use of the product, the brakes and wheels should be checked.
- Check the stability of the chair before use.
- Only use on level ground.
- When stationery, apply the brakes on the wheels.

- Ensure the brakes are released before the chair is moved again.
- Always ensure that the device is stationary and the brakes are applied when transferring to and from the device.
- Do not lean out of the device.
- Check the wheels bearings the brakes
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING & USE

After unpacking the box, first remove the plastic cable tie and the packaging bubble bag with scissors, in the meantime paying attention to keep all the spare parts and accessories in the box from damage. If you conduct the assembling on the ground, first lay a piece of paper or carpet on the ground to prevent scratching, also remembering to distinguish the spare and accessory parts from the subassemblies, and then remove the inner carton to take out the parts and place them in good order. What's contained in the box is as shown in Figure 1:

1. Cloth basket Assembly
2. T-shape Handle Assembly
3. Frame Assembly
4. Tool kit
5. Eight-inch Wheel Assembly
6. Foldable Upper Bar Assembly
7. Knee Platform Assembly
8. Spare and Accessory Parts

Figure 1

As shown in Figure 2 below, the tool kit in :

- A1: a 5mm Allen Wrench
 A2: a 6mm Allen Wrench
 A3: a Hexagonal-hole Spanner

Figure 2

As shown in Figure 3 below, the loose pack of spare and accessory parts:

- B1: Carriage Bolt M10*58
- B2: Outer Hexagonal Screw M8*45
- B3: Standard Hexagon Socket Head Cap Screw M6*35
- B4: M8 Screw with Star-shaped Knob
- B5: Iron Dust Cover
- B6: M8 Self-locking Nut
- B7: M10 Knob Nut (Star-shaped)
- B8: Velcro strap
- B9: M6 Self-locking Nut
- B10: $\phi 8.5\text{mm}$ Flat Washer
- B11: $\phi 8.5$ Spring Washer
- B12: M8 nut cap
- B13: $\phi 6.2$ Flat Washer
- B14: Standard Hexagon Socket Head Cap Screw M8

Figure 3

(Please refer to the part number and part name of Figure 3 for assembly)

Step 1 :

Take out the B3, the B13 and the B9 and use them to fix the joint bearing assembly as shown in Figure 4.

Step 2 :

As shown in Figure 5, first align the foldable upper bar assembly with the hole on the lower main frame of the knee walker and then lock it with the B1 and the B7, and then press down the spring quick release to fasten

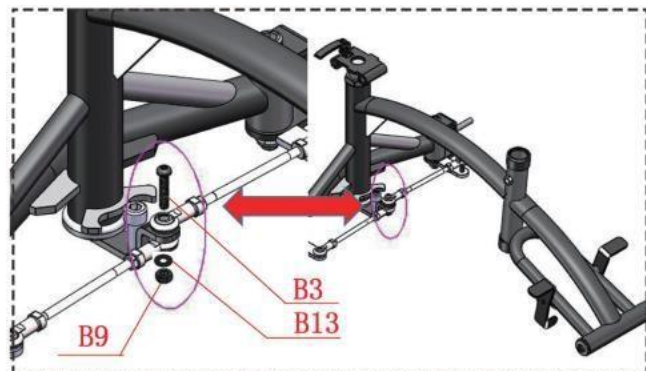
Figure 4

Figure 5

Step 3 :

Insert the T-shape handle bar into the foldable upper bar assembly as shown in Figure 6, and then screw the B4 and B2 into the corresponding hole to lock the connection

Figure 6

Step 4 :

Assemble the front and rear wheel assemblies according to the parts orders shown in Figure 7 with reference to Figure 3 for the assembly of the parts

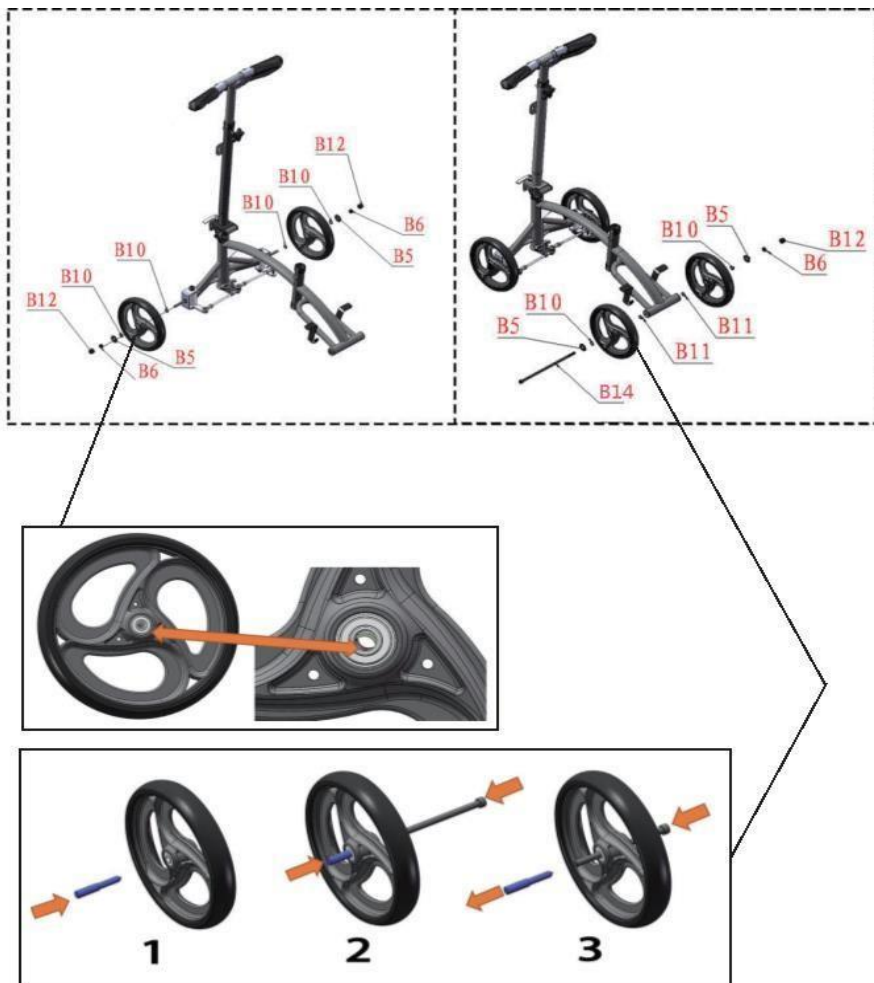
Figure 7

Step 5 :

Tie the brake cable with a Velcro strap, as shown in Figure 8.

Step 6 :

Distinguish the front from the rear before installation, as shown in Figure 9. Align the knee platform with the adjustment tube hole of the walker frame, and lock this position with one B4 and B2. The installed knee platform should basically keep level with the

Step 7 :

As shown in Figure 10, put the hooks on the cloth basket through the two holes on the foldable upper rod assembly and lock

Figure 8

Figure 9

Figure 10

1. Check whether the installation of the knee walker is corrected by referring to the specification drawing for comparison
2. Verify if all accessories of the walker are fully assembled and if there are unnecessary accessories.
3. Check if the walker is loose or not. If it is loose, tighten the screws and nuts.
4. Check all the components of the walker by its appearance to see if there are any skewings, and if any, make appropriate adjustments.

Instructions on some components :

1. Brake Operation: As shown in the figure below:

D1: Normal use state, brake disengaged.

D2: Engage the brake as needed in the use process of the walker.

D3: Press down the brake lever to keep braking state.

2. Height adjustment of the handle assembly and knee platform assembly :

Please unscrew the B2 and B4 to adjust them to the desired height, and then lock in place, as shown in figures D4 and D5 :

3. As shown in Figures D6 and D7 : the tightness of the brake can be adjusted by adjusting the handle nut and brake nut.

4. When the bushing inside the wheel assembly is not concentric with the axis, a screwdriver or a hex wrench can be used to adjust it to be concentric with the axis and pass through a M8 screw to make it tightened, as shown in Figure D8/D9/D10.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean by hand the product (all wheels, seats, and fixations) with a damp cloth and a mild detergent and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- The walker must be completely reviewed every year, with compulsory lubrication.
- It is recommended that the device is serviced annually.
- If the device is subject to heavy use and/or use in a multi user environment then we recommend the device is serviced more frequently.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
 - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
 - All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

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GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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