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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Neo Strong rollator is a walking aid for people with reduced mobility. This rollator is designed for indoor and outdoor use and folds up for easy storage or transport.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 150 Kg.
- Do not exceed the maximum load limit in the basket (5kg). Only transport loads in the basket. Do not hang heavy objects on the rollator as this could affect its stability.
- Make sure that all parts (wheels, cables, handles and knobs) are secure and in good working order.
- Do not use the rollator on stairs or escalators.
- Only use accessories and parts supplied by the manufacturer.
- The rollator is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may damage the wheels and the structure of the rollator itself. The rollator could be weakened and no longer meet its mechanical characteristics in future use. Only use the rollator as a walking support.
- Do not use the rollator to walk backwards.
- Do not make any adjustments to the rollator while it is in use.
- Check that the brakes work properly before use.
- All wheels must always be in contact with the ground.
- When the rollator is in the stationary position, the brakes must be locked. Do not attempt to move the rollator if the brakes are locked.
- Be sure to put your hands on each handle so that the rollator can be in the parallel force position.
- Use caution when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Do not use the walker on a wet, icy, or snowy slope.
- If any parts are damaged, broken or worn, contact the dealer.

PRODUCT DESCRIPTION

1 : Handle (x2)	7 : Front wheel (x2)
2 : Handle tubing (x2)	8 : Cane holder (x1)
3 : Brake handle (x2)	9 : Bag (x1)
4 : Knob (x2)	10 : Backrest (x1)
5 : Seat (x1)	11 : Brake cable (x2)
6 : Rear wheel (x2)	12 : Reflective sticker (x2)

UNFOLDING :

Place the rollator on its wheels and press down on the two side posts of the seat until they click into place (see 1 and 2 below).

BAG INSTALLATION :

the bag is attached to the ends of the frame with velcro.

HANDLE HEIGHT ADJUSTMENT :

Insert the handles at least up to the first through hole in the frame tube. Push the fixing screw through from the inside and tighten it with the washer and the knob. To change the handle height adjustment, loosen the fixing screw by turning the knob counter-clockwise. Then lock the handle at the desired height as described above. (See 3 and 4 below).

CANE HOLDER INSTALLATION :

Under the backrest and on the handle posts, insert the cane holder by inserting the lugs into the holes, close the clip, press firmly until it clicks into place. Insert the cup with its stem at the bottom into the hole provided and clip it in place. Check that it fits properly. The cane holder can be mounted on the right or left.

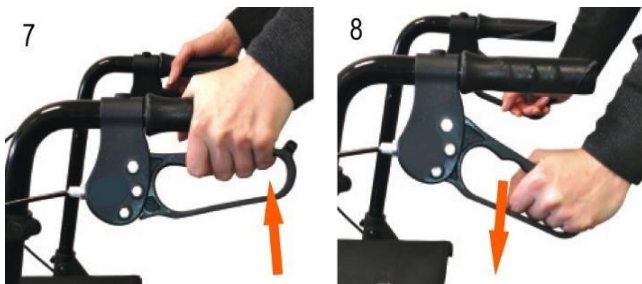
BRAKE ADJUSTMENT :

If the braking is too weak or too abrupt, the brake can be adjusted to the desired tension either above, on the handle (see 5 below), or below, with a suitable tool (not supplied) at the adjustment screw on the cable (see 6 below).

USE OF BRAKE HANDLES :

To slow down, squeeze the handles slightly with your fingers. Squeeze the handles simultaneously on the right and left (see 7 below). To lock the brakes, for example to stop the rollator, push both handles down firmly (see 8 below) until they lock. A "click" will be heard, indicating that the lock is secure. To release the brakes, pull up both handles (see 7 below).

The action of the brakes can be influenced by tyre wear. **It is therefore necessary to check the brakes regularly.**

SITTING POSITION :

Place the rollator on a flat, stable surface. Then **lock the brakes** in the parking position as described above. The rollator should be stable and not move. Sit upright and do not lean back.

FOLDING :

To fold the rollator, pull the seat up using the handle and fold the rollator to either side. (See 10 and 11 below).

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.

- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width. 60,5 x Depth. 67 x Height. 81 / 96 cm
Seat dimensions	Width. 40 x Depth. 22,5 x Height. 58 cm
Width between the handles	47,5 cm
Maximum bag load	5 kg
Weight	7,9 kg
Maximum supported weight	150 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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