

<u>User Instruction for the Utility Over-bed/Chair Table</u> <u>Mobile – B59 & Static – B61</u>

1. Introduction

The Utility Over-bed/Chair Tables can be packed flat for easy transportation and storage. Powder coated finish and constructed from mild steel, and adjustable in height and tilt.



Product specification	
Table top size	610mm x 405mm
Adjustable table height	675mm – 1045mm
Footprint	610mm x 405mm

Parts list	
Description	Part No
Castors (Set of 4)	70023I/1

2. Contents

- Over-bed/Chair Table (Various)
- User Instructions (this leaflet)

3. Assembly Instructions

- Carefully unpack and examine the product.
- Before use, check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If any signs occur
 please contact your supplier to exchange and refrain from using the product. DO NOT ATTEMPT TO
 USE IF PRODUCT IS DAMAGED.

4. Correct Use

- The table is designed to help the user by being used as a dining, reading, or writing aid.
- The table is not intended to support full body weight so refrain from attempting to lift full body weight via the product.
- The table is designed to be used in conjunction with either a bed or chair.
- Do not attempt to overload the table.

Adjustable Height:

- Unscrew both hand wheels
- Raise/lower to required height
- Retighten the hand wheels
- Tilt table top to angle required by raising or lowering on each side of the table
- Tighten both hand wheels to lock top into desired position.

5. Cleaning

- Use mild detergent with warm water to wipe down framework on at least a weekly basis more frequently if required.
- Do not use an abrasive cleaner as this will damage the frame and nullify your warranty.



Ref: B59 & B61 April 2014 Form No: Cef 0035



<u>User Instruction for the Utility Over-bed/Chair Table</u> Mobile – B59 & Static – B61

• The beech table top is not suitable for machine cleaning and should be removed and cleaned as above if the frame is to be cleaned by mechanical means.

Use of power/jet washing & industrial washing machines:

- If either of the above is used to clean equipment care must be taken since high-pressure water can be penetrative.
- Care should be taken that frames are not placed too close to high-pressure water jets.
- After cleaning ensure any residual water is removed from the framework as it could cause permanent
 rusting of the frame (leave frames open in a warm environment for several hours to ensure all internal
 moisture evaporates).
- We cannot offer advice on industrial washing machines with regards to specific machine settings, as these will be dependent on the manufacturer's instructions.

6. Maintenance Requirements

• The product should be checked on a regular basis for damage or wear and tear – this includes flaking or cracking paint. Any undue signs of damage should be noted and reported to the supplier in the first instance. Should damage be evident, then the user should refrain from using the product until advice has been sought from the supplier.

7. Warning

- This product is intended for use as a dining, reading or writing aid it does not support heavy weights.
- We recommend that a full risk assessment should be carried out before use.

8. Warranty

This product carries a 12 month warranty from the date of purchase against manufacturing defects. Misuse or damage is not covered. This warranty does not affect your statutory rights. As manufacturers we cannot specify expected product life as this is based on the level of usage.

9. Disposal Instructions

- Appropriate hygiene standards should be applied before disposal to reduce the risk of infection.
- Materials used are essentially non hazardous and can be disposed of appropriately.

10. Contact Details

We have attempted to cover most eventualities, however if you have any specific queries, please contact:

Cefndy Healthcare Cefndy Road Rhyl Denbighshire LL18 2HG

Tel: 0044 1745 343877 (Quality Assurance Dept) E-mail: cefndy.sales@denbighshire.gov.uk

