





When lives hang in the balance, you need a CPAP system that lives up to the challenge. The Flow-Safe II EZ® is a disposable CPAP system with integrated nebulizer requiring only one gas source. Flow-Safe II EZ® helps give you an advantage in life-saving emergencies.




## **Advantages**

• Clinicians can now deliver aerosol & CPAP therapies with just one integrated oxygen source.

- Consumes 50% less oxygen than the original Flow-Safe® (Increased flow may be required when using the nebulizer)
- Integrated nebulizer.
- Built-in manometer & pressure relief valve.
- · Advanced mask design.



The Flow-Safe II EZ® conserves oxygen while maintaining high FiO₂ delivery. Saves resources and represents a major advantage on long transports. Increasing flow may be necessary when activating the nebulizer.

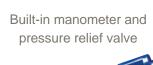





## **Clinical Use**

- By connecting the Flow-Safe II EZ® CPAP System to a flowmeter/regulator and adjusting flow rates up and down, various pressures can be achieved.
- With the nebulizer switch in the red position all of the oxygen flow goes through the CPAP valve.
- With the nebulizer switch in the green position approximately 6 LPM of gas flow is diverted to the EZflow MAX® continuous nebulizer.
- Yellow Flow/CPAP pressure chart on all devices.
- To achieve the desired CPAP pressure, adjust oxygen flowrate according to the chart (high capacity flowmeter/regulator may be required).
- With nebulizer on, the flow rate must be adjusted up to maintain the same CPAP pressure.



Over 55 Years of Medical Device Sales and Development

Dedicated to delivering clinically differentiated critical care technology that saves lives throughout the world.

(800) 835-6633 www.mercurymed.com