TurnAid Slide sheet

Material: 100% Nylon



Use:

TurnAid Slide sheet is used together with TurnAid turning system T3. Slide sheet 95 cm fits an 80-90 cm wide mattress.

Placement of slide sheet:

The TurnAid slide sheet is mounted directly on the mattress. Fold out the Slide sheet on top of the mattress and place it with the seams downwards.



Place the elastic bands at the Slide sheet corners on the 4 corners of the mattress. Straighten out the surface of the slide sheet, so it doesn't folds and curls.



Close the 4 Velcro straps around the bed's <u>adjustable</u> <u>surface parts.</u>

If the straps are wrongly attached to the bed's frame or the motor rolls, the straps and possibly the system together with the bed will be damaged, when the bed's surfaces are raised.

The straps are long enough to fit also extra thick mattresses. Cut off what is left over of the Velcro straps after closing them.

Cleaning:



95° Machine wash



Tumble dry at normal temperature



Do not iron

Do not use fabric softener.