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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

The Evolution 2 Wheeled Rollators are walking aids for people with reduced mobility. These products are intended for use outdoors and can be folded up for easy storage or transport.

PRODUCT DESCRIPTION

1. Handgrip
2. Adjustable tubing handgrip
3. Brake lever
4. Interlock button for handgrips
5. Backrest
6. Seat
7. Bag Hook
8. Front wheel
9. Brake adjust screw
10. Rear wheel
11. Safety clasp
12. Kerb climber
13. Crutch holder
14. Release button
15. Cup Holder

The rollator includes:

- Backrest
- Crutch holder
- Cup Holder
- Instructions manual

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 136 Kg.

Do not exceed the maximum load limit in the basket (5kg). Loads must only be transported in the basket. Do not hang heavy objects on the rollator as this could affect its stability.

Make sure that all parts are securely fastened and in good working order.

Regularly check the screws and fasteners of the rollator and retighten if necessary.

Make sure that the handle locking latch is securely engaged.

Make sure that the rollator is fully opened and seat is down and the folding bar under seat is locked in the down position.

Be sure that both handgrips are fully tightened and are at equal heights. Test the rollator for stability before applying the full weight to the handgrips.

Do not use the seat when the rollator is on sloping or uneven ground. Use only on even and solid surfaces. Replace the wheels when they are badly worn or damaged.

DO NOT MOVE THE ROLLATOR WHILE SEATED. This is a walking aid only and is not to be used as a transportation device.

Test the brake lever before operating the Rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning.

When locks are released, the wheels should turn freely.

Always lock the brake lever before sitting and do not use seat when unit is on an incline or uneven ground.

Replace tyres when significantly worn or damaged.

Do not use the rollator on stairs or escalators.

Use only accessories and parts supplied by the manufacturer.

Do not use the rollator for walking backwards.

Do not make any adjustments to the rollator while it is in use.

All wheels must always be in contact with the ground.

Use caution when climbing pavements or walking on inclined, uneven, or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.

Do not use the walker on a wet, icy, or snowy slope.

If any parts are damaged, broken or worn, contact the distributor.

MOUNTING & USE

Detachable rear legs

With detachable rear legs (Figure 1), it provides compact packing and convenience for transport. By pushing the release button underneath the rear tube (Figure 2), easy to detach the rear legs. Also, same way to assemble the rear legs before use (Figure 3).

Unfolding the rollator

Open the safety clasp and pull the operating handles apart (see figure 4). Press the side bars down until you hear a small click in the fully folded position (see figure 5). Before using the rollator, make sure that the folding mechanism is locked in place.

Height Adjustment

Lift the catch and push or pull the handgrip to the correct height. Press down the catch, the handgrip will now be locked at the correct height. (see figure 6)

Removing/Attaching the carry bag

To attach the bag strap onto bag hook on the front end of the seat bar facing outward (see Figure 7 & 8). The rollator bag is available for purchase.

Braking/ Locking

Pull both brake levers upwards to apply the brakes while the rollator is moving (see figure 9). Always brake slowly and steady. Press both brake levers down to apply the locks while the rollator is stationary (see figure 10). Pull up the brake levers and release to release the locks. The effect of the brakes can be affected by abrasion of the wheels. Regularly test the brakes.

Adjustment of brakes

To adjust the braking strength, pull up the brake cable (see figure 11) adjust the brake by pressing the nut upwards or downwards (see figure 12)

Sitting

Position the rollator on an even and solid surface. Lock the brakes (parking position). The rollator must stand and must not slip. Sit straight up and do not lean backward. (See figure 13)

DO NOT MOVE ROLLATOR WHILE SEATED ;

Folding/Storage

To fold, pull seat upwards by the handle on the seat plate as shown in (see figure 14). The rollator can now be folded completely and secured by using the safety clasp as shown in (see figure 15).

Kerb climber

We recommend you use our built-in, kerb climber (see figure 16). This saves you lifting the rollator over obstacles such as thresholds and kerbs. Just step down on the kerb climber and with a gentle pull back on the handles, the rollator will raise its front wheels to easily maneuver over obstacles.

Crutch holder

The elegant crutch holder consists of two parts, an upper hook and lower foldable base, and is used to take along a crutch or a walking stick with the rollator.

Cup Holder

The rollator comes with a cup holder, which is ideal for transporting drinks.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminum and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.

- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.

- **Checklist :**

To keep the rollator in good condition, check before each use, and check regularly as follows :

Check the item	Anytime	Weekly	Monthly	Yearly
Clip and knob for height adjustment		X		
Structural weld at wheels		X		
Cleanliness	X			
Screws and fasteners		X		
Stability of frame and backrest			X	
Condition of seat				X
Brakes	X			

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable, and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your rollator for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your rollators.

- In a dry and temperate place
- Protect your rollator by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the rollator elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

Overall Dimensions	Width 69 cm x Depth 71 cm x Handle height 87 to 101 cm
Folded Dimensions	Width 32 cm x Depth 75.5 cm x Height 88.5 cm
Seating Dimensions	Width 46 cm x Depth 21 cm x Height 59 cm
Distance between grips	45 cm

Weight	8.2 kg net
Max. User Weight	136 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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